

Caution

- Be sure to hold the handpiece firmly to prevent the handpiece from slipping out of hand by the spray pressure when lubricating.
- Supply lubricant until it comes out of the handpiece head (for approx .2 seconds).
- Hold the Lubricate spray can upright.

- (2) Lubricant oil
 Swing open the latch plate. Loosen the Union Nut on the head.
 Remove the head from the sheath. Remove the Drive Shaft assembly from the head or sheath.
 Supply lubricant at the arrows.(Fig. 4)

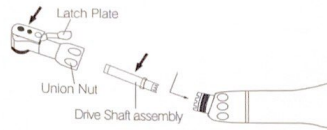


Fig.4

5. Cleaning and Sterilization

The products listed below are washable with Thermo Disinfectant.
 ECM, ER4M, ER10M, ER16M, ER64M

Steam autoclave is recommended.
 Sterilization is required first time you use and after each patient.

Autoclave Procedure .

- 1) Scrub dirt and debris from the handpiece, and wipe clean with alcohol-immersed cotton swab or cloth. Do not use a wire brush.
- 2) Lubricate the handpiece. Refer to 4. Lubrication.
- 3) Insert into an autoclave pouch. Seal the pouch.
- 4) Autoclavable up to max.135°C.
 ex.) Autoclave for 20 min, at 121°C or 15 min. at 132°C.
- 5) Keep the handpiece in the autoclave pouch to keep it clean until you use it.

* Sterilization at 121°C for more than 15 minutes is recommended by EN13060 or EN554.

Caution

Do not wipe nor immerse the handpiece in acidic water or acidic solutions.

6.Cartridge Replacement NAC,BB,NBBW)

- 1) Loosen the Union Nut and remove the head from the sheath.
- 2) Pull out the Drive Gear Assembly from the head.(Fig.5)

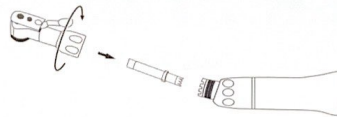


Fig.5

- 3) Place Cap Wrench on the End Cap and turn counter-clockwise. Unscrew and remove the cap. (Fig. 6)

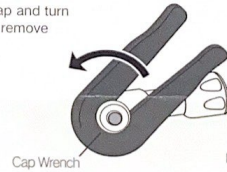


Fig.6

- 4) Swing to open the latch plate. Insert a bur blank or round rod into a hole from the head end, and push out the cartridge.
- 5) Insert the new cartridge, with its pin aligned in the slot of the head. (Fig. 7)

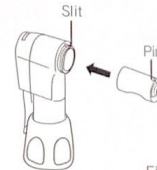


Fig.7

- * Optional Cartridge : NAC-O3 (Cartridge for NAC-Y) Order No. C030 011
- BB-O3 (Cartridge for BB-Y) Order No. C037 015
- NBB-O3 (Cartridge for NBBW-Y) Order No. C210 015

CE The EU directive 93/42/EEC was applied in the design and production of this medical device.

NAKANISHI INC.
 www.nsk-inc.com
 700 Shinohinaka Karunuma-shi
 Tachigi 222-8666,
 Japan

NSK Europe GmbH
 www.nsk-europe.de
 Ely-Berthum-Strasse 8
 65160 Eschborn,
 Germany

NSK United Kingdom Ltd
 www.nsk-uk.com
 Office 5, Gateway 1000,
 Avington Business Park, White Way,
 Stevenage, SG1 2EP, UK

NSK America Corp
 www.nsk-inc.com
 700 Cooper Court
 Schaumburg, IL 60173,
 USA

NSK France SAS
 www.nsk.fr
 19 avenue de Villers
 75017 Paris,
 France

NSK Dental Spain SA
 www.nsk-spain.es
 C/Adriana, 43 El Soto-Estipolis
 28232 Las Rozas, Madrid,
 Spain

NSK Oceania Pty Ltd
 www.nsk-inc.com
 Unit 22, 198-222 Young St.
 Waterloo, Sydney
 NSW 2017, Australia

NSK Middle East
 www.nsk-inc.com
 Room 5EA-701,7th Floor, East Wing, No.6
 Dubai Airport Free Zone,
 PO Box 54316 Dubai, UAE

NSK Asia
 www.nsk-inc.com
 1 Maritime Square,
 #09-33 HarbourFront Centre,
 Singapore 099293

Specifications are subject to change without notice.

09.12.10